**+Miofresh**  
After

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Owing to these multifunctional elements combined, we have been able to create the Medical Device Class I, Miofresh After.

It is a Hydrogel sticking plaster that, when it is applied, doesn't cause side effects of any kind. However, at the same time, it guarantees all the benefits we expect during its application.

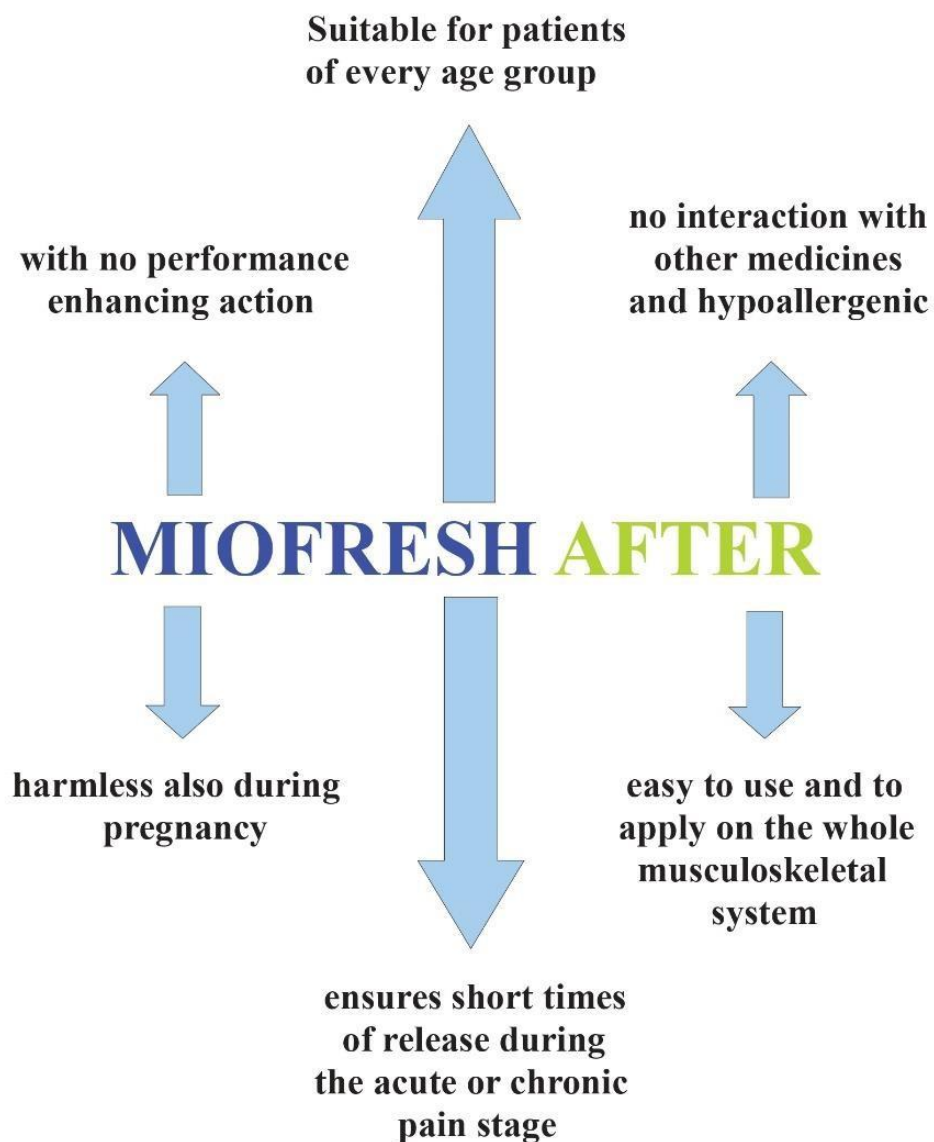
This is possible for its natural privileged action, result of studies and research in the health-medical area, on the quality of the vegetal molecules and on the production of the structure where they are included.

For its features, this device is suitable for every age group, as it can be easily used on the whole musculoskeletal system.

Actually, its action helps to reduce the upsetting drawbacks caused by traumas, rheumatic, articular, edematous and muscular pains.

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
As indicated, we are going to show the results of the laboratory research and tests, carried out at the *Università degli studi di Pavia*. These actually confirm that the medical device have effect and guarantees, in a short time, the chance to relieve the pain during its acute stage and during the chronic one. Thus, it safely reduces the inflammation. The following is the first of the tests we carried out, in a chronological order: alteration for the skin.

Alteration for the skin

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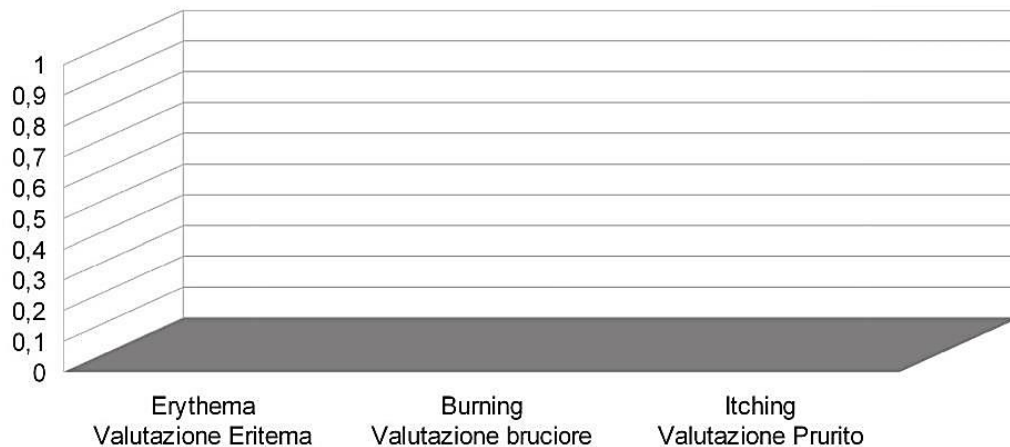
Sensation of coolness in the application area

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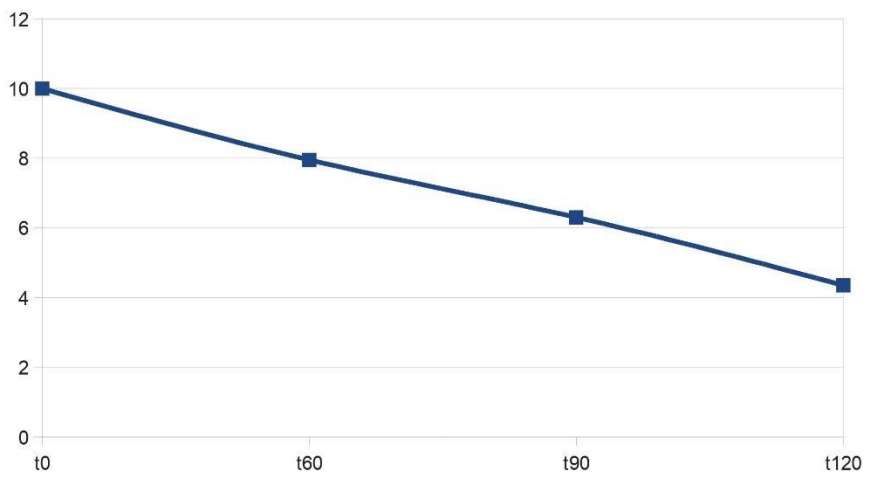
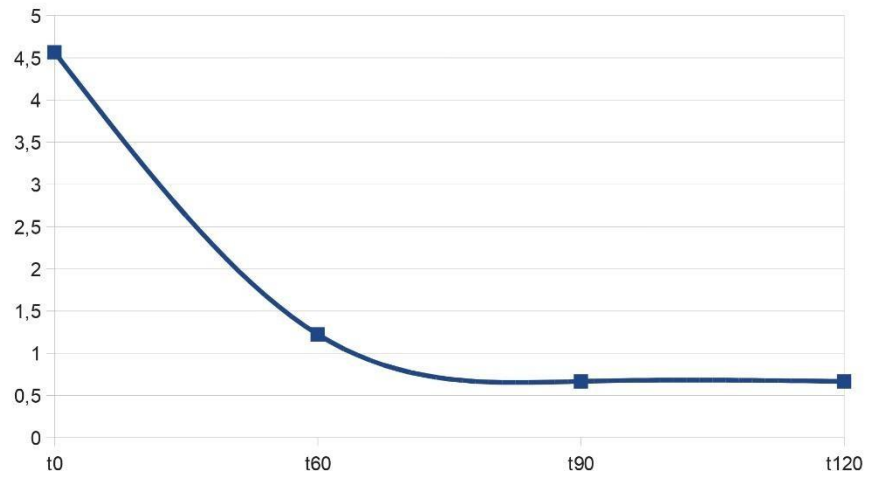
The graph below shows the reduction of the pain in the short time of the application.

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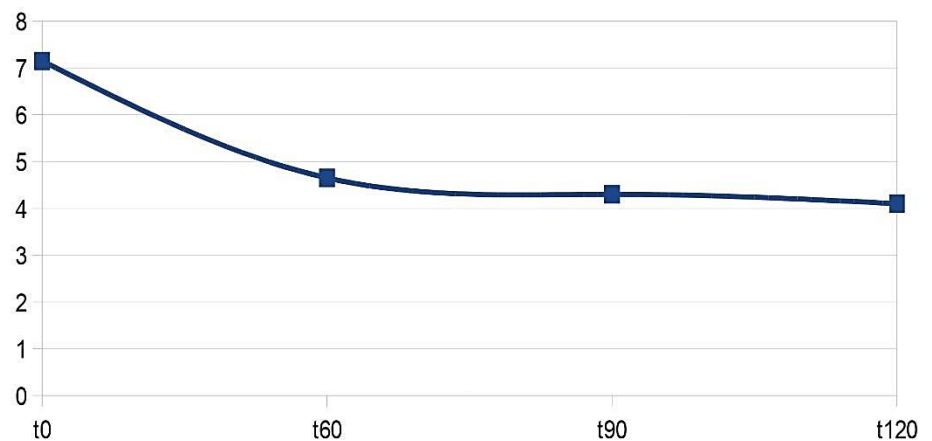
The anti-inflammatory activity, with the application of the sticking plaster, also helps to reduce the pre-existing swelling.

### Reduction of the pre-existing swelling

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
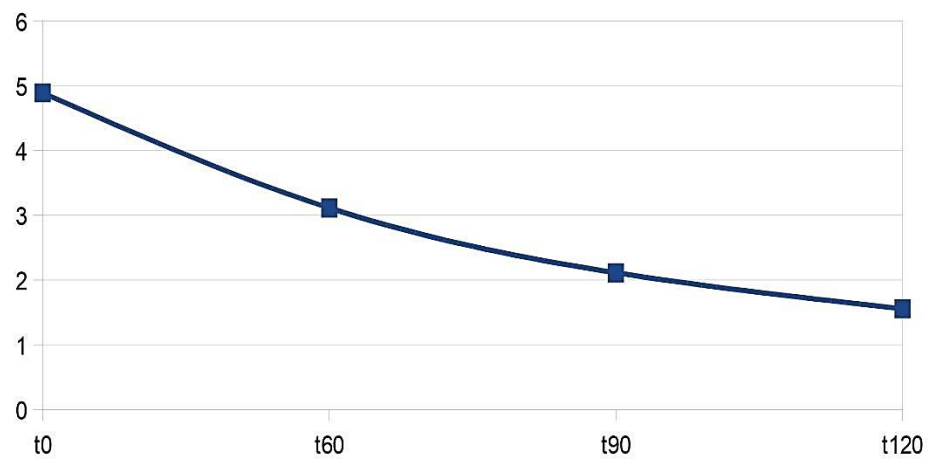
Adhesiveness

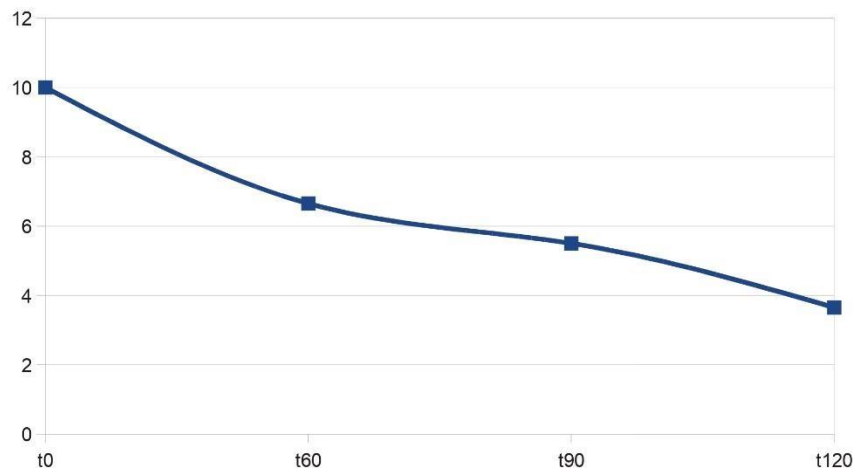Tenuta adesivo

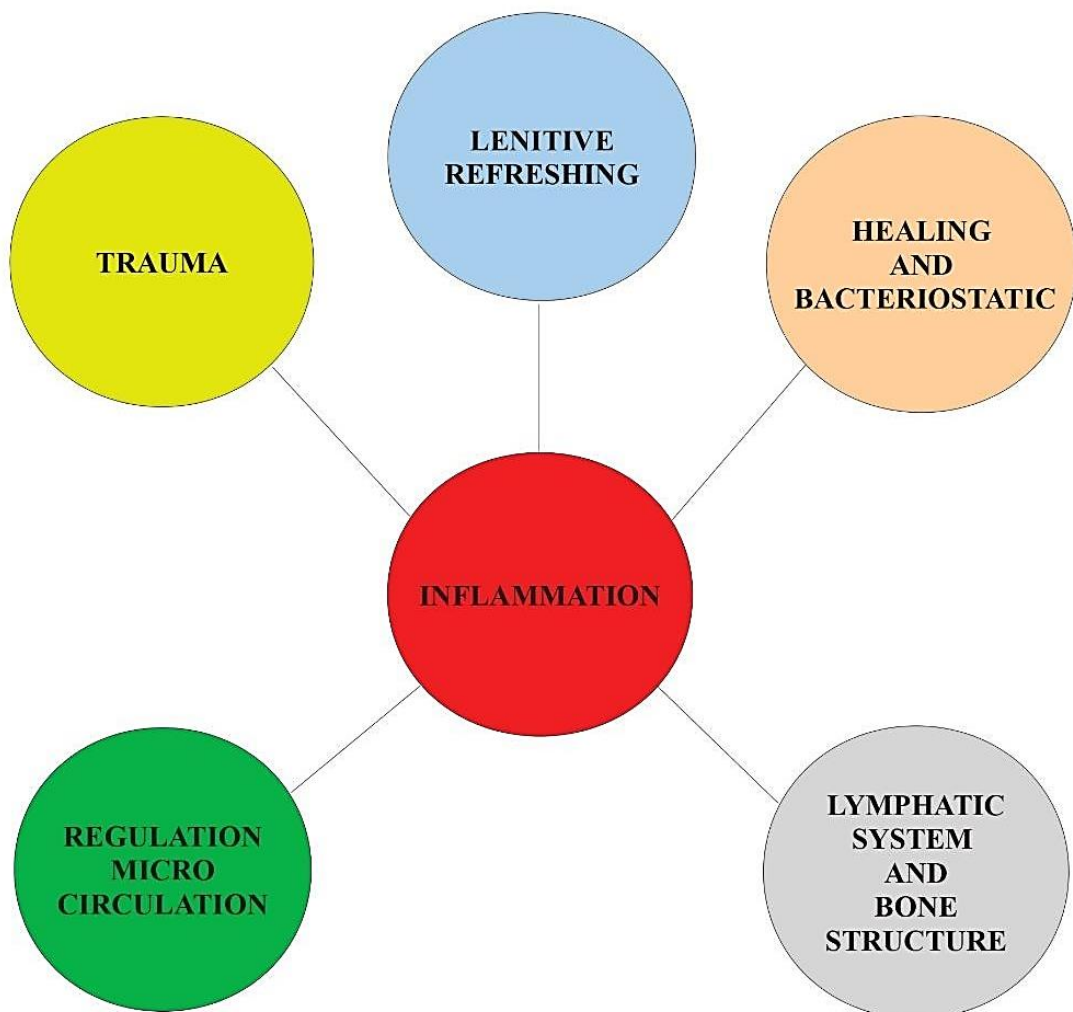After showing the graphs, with the respective projections at the Università di Pavia, we are going to consider the structure and the technological Know-how, with the formulation of the phytotherapeutic elements of the medical device:

**Indications:**

It is recommended for its anti-inflammatory, soothing and refreshing properties with analgesic, anti-neuralgic, soothing and refreshing action for the musculoskeletal system, the ligatures and the tendons and the osteoarticular system, with anti-rheumatic action.

Since it is versatile and hypoallergenic, it can be safely used during pregnancy, in insulin-dependent, immunosuppressed patients, with cardiovascular risk or cancer. Patients may also continue their pharmacological therapies.

As it is free from any performance-enhancing effect, it can also be used in sports activities for traumas and rehabilitation.

**Dosage:**

It is recommended to apply the plaster twice a day, every 8 hours on the injured part. It can be repeated for more days.

**Warnings:**

The medical device Miofresh after is a controlled-release plaster that continues its action for 8 hours after it has been removed. It must be applied only on the parts of the body without abrasions.

**PHARMA 1 HUMANITAS IT IS THE  
WHOLESALE DISTRIBUTOR OF  
THE MEDICAL DEVICES SUPPLIED  
FROM OUR ITALIAN PARTNER  
GROUP.**

[www.pharma1humanitas.com](http://www.pharma1humanitas.com) -[pharma1humanitas@gmail.com](mailto:pharma1humanitas@gmail.com)